



## Intersurgical i-flo™ High flow nasal cannula



**Critical Care** ▪ i-flo - High flow nasal cannula



## Intersurgical i-flo™ - High flow nasal cannula


The i-flo cannula is Intersurgical's single use patient interface designed for the delivery of High Flow Oxygen Therapy (HFOT). The i-flo cannula is available in three sizes and is for use with adult patients in a hospital environment.

HFOT has become a key mode of respiratory support in patients with hypoxemic respiratory failure and is considered to offer a number of benefits versus more conventional oxygen therapies. These benefits have been shown to include:

- Improved oxygenation
- Reduced work of breathing
- Improved secretion clearance
- Improved patient tolerance
- Avoidance of intubation<sup>[1]</sup>

The i-flo cannula complements Intersurgical's extensive range of Critical Care breathing systems and interfaces and the unobtrusive design has been developed with a focus on performance and patient comfort.


The i-flo includes a number of unique features and benefits to help ensure the optimal delivery of high flow oxygen therapy.

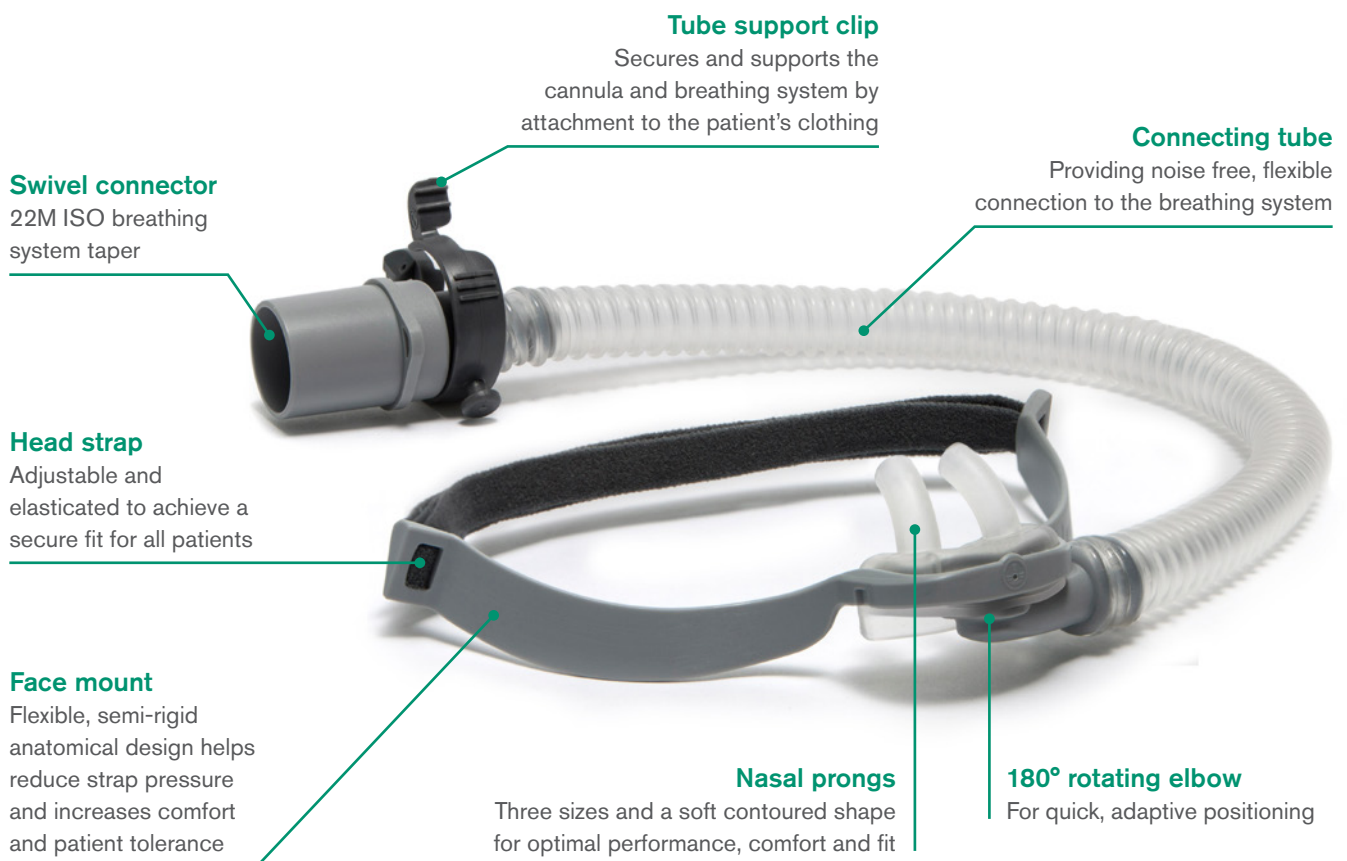

[1] Masaji Nishimura. High-Flow Nasal Cannula Oxygen Therapy in Adults: Physiological Benefits, Indication, Clinical Benefits, and Adverse Effects. Resp.Care 2016 Apr;61(4):529-41



## Adaptive and secure

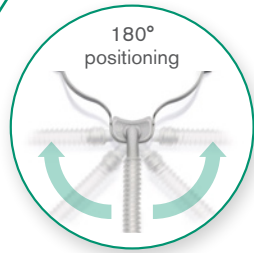

The i-flo's unique, 180° swivel elbow enables easy and convenient positioning of bedside equipment.

The i-flo clip and lanyard tube management system securely supports the breathing system tubing and allows the patient to move freely without concerns of cannula dislocation or discomfort.

## Comfortable and supportive



An anatomical face-mount design directs the head strap above the patient's ears, helping to minimise rubbing and discomfort.

The flexible, semi-rigid face-mount also ensures that strap pressure is distributed evenly over the patient's cheeks. This increases cannula stability and support and helps to prevent pressure hot-spots from developing.





## Intersurgical i-flo™ cannula

Code	Description	Box Qty.
7300000	i-flo high flow nasal cannula, small adult	7
7301000	i-flo high flow nasal cannula, medium adult	7
7302000	i-flo high flow nasal cannula, large adult	7

Make an enquiry





▶ i-flo setup video is available at [www.intersurgical.com](http://www.intersurgical.com)




## Single limb breathing systems


Code	Description	Box Qty.
2060310	22mm Flextube single heated wire limb with auto-fill chamber and extra limb. ≥ 1.6m	7

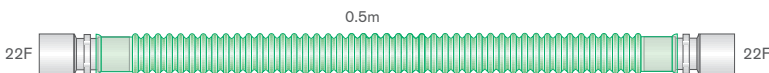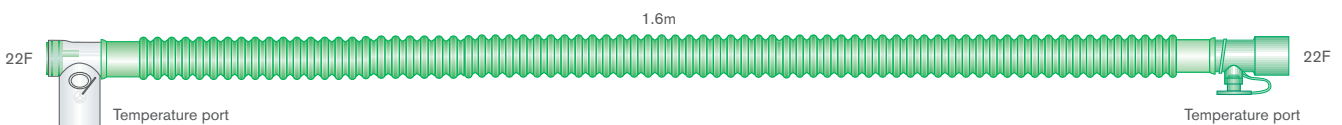

Humidification chamber

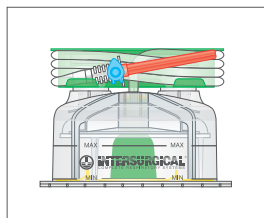

Tube management clip

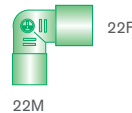

Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK

T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 [info@intersurgical.com](mailto:info@intersurgical.com) [www.intersurgical.com](http://www.intersurgical.com)



The manufacturer Intersurgical Ltd is certified to ISO 9001:2015, ISO 13485:2016 and ISO 14001:2015

**Please think before you print**  
Save energy and paper.  
If you must print this information sheet please print it double sided.

IS1.53 • Issue 1 12.20