

Pre use check: DuoFlow™ bi-lumen breathing system




1 Attach the expiratory limb of the system to a gas flow.



2 Remove all patient connections.



3 Occlude the expiratory side of the breathing system at the patient end using the supplied leak tester tool.

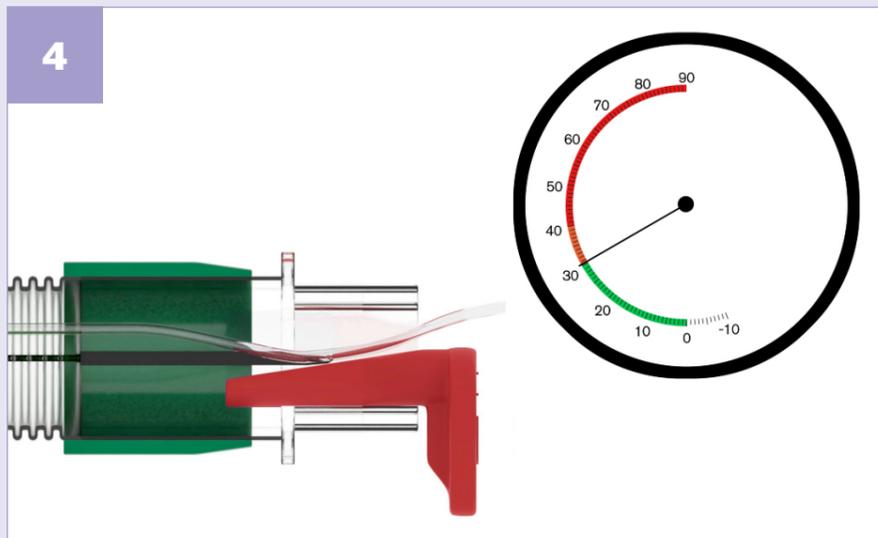

4 Pressurise the expiratory limb to 30cm H₂O and confirm that there is no leak or pressure decay.



5 If no pressure decay is detected, confirming that the inner wall is leak free, attach the breathing system to the ventilator/anaesthetic workstation and complete all pre-use checks as recommended in the instructions for use of the ventilator/anaesthetic workstation ready for clinical use.



6 A leak test of the complete system and all accessories must be performed before use.