


Universal Stylet Bougie (USB™)

A hybrid stylet and bougie in one simple to use device


The unique design of the USB incorporates two malleable sections with different and distinct properties either side of a more flexible section. The USB can easily be manipulated to a variety of angles when used as a stylet, yet has the flexibility when used as a bougie to provide a significantly enhanced level of tracheal ring feedback.

Key features

Low friction hexagonal shape
for optimal insertion and removal

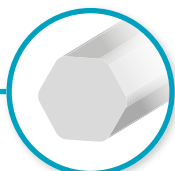

Device information
confirmation of size, diameter and other key product details

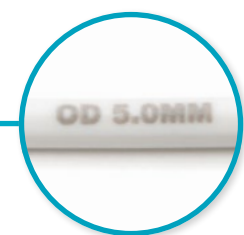

Flattened (non-patient) end
allows the user to insert the end of the USB back into the connector of the tracheal tube to retain product position

Coudé tip
to optimise insertion. Atraumatic to reduce the potential for trauma

optilube™
a sachet of lubricant is included in each pack to facilitate optimal insertion and removal (8090015 only)



Malleable sections and mechanical amplification
the sections of the device marked in grey indicate where the USB is malleable, allowing manipulation of the USB to the desired angle, and in combination with the flexible middle section, provide enhanced tracheal ring feedback

Length markings
confirmation of length at 45, 20 and 12 cm from the tip of the patient end of the device for correct positioning



Code	Description	Box Qty.
8090015	Universal Stylet Bougie with optilube™	10
8080015	Universal Stylet Bougie	10

[Find out more](#) [Watch the video](#)

The USB can be used in a variety of ways as both a bougie and a stylet:

As a bougie:



As a standard bougie, with the USB straightened prior to use/insertion.



As a standard bougie, but with the USB retained in its original shape. The user can insert the device whilst still in this shape to retain directionality, then railroad the tube over the USB once correctly positioned.



The Tracheal Tube can be pre-loaded over the USB so the tip of the tube is located at the point where the second malleable section begins (identified by the grey sections). The proximal end is then folded back over into the Tracheal Tube so the tube does not move accidentally. Once the bougie has been correctly positioned, the Tracheal Tube is railroaded over the USB as normal.

As a stylet:



Straight to cuff.



Curved.

