



STARMed Ventumask Basic

For CPAP therapy with Venturi flow driver and adjustable PEEP valve



Now available

with nebuliser port option

CV209S (Small)

CV209M/L (Medium/Large)

CV209XL (Extra Large)

Nebuliser kit not included

Critical Care ▪ Patient Interfaces ▪ STARMed

STARMed Ventumask Basic with Venturi flow driver & adjustable PEEP valve

StarMed Ventumask Basic


StarMed Ventumask Basic is a unique design for CPAP therapy that is simply activated by connecting to an appropriate oxygen supply. It can be used in a hospital environment or pre-hospital use, for example in an ambulance.

Advantages and Technical Features

- Ventumask Basic is a complete and easy to use system, where the oxygen supply tube, a Venturi flow driver and a PEEP valve are integral to the CPAP mask
- The integrated Venturi flow driver is operated by a single oxygen flowmeter to generate high patient flows while consuming a reduced amount of compressed gas
- The built-in adjustable PEEP valve is designed exclusively for Ventumask Basic and the administration to the patient is facilitated at some pre-settings: 5 - 7.5 - 10 - 12.5 cmH₂O
- O₂% of 40% can be delivered at each PEEP level
- The user is guided by a performance table integral to the product, to be aware of patient flow at each PEEP settings

The kit is composed of:

- CPAP mask with safety valve and port to monitor actual O₂% or pressure
- Venturi flow driver integral to the mask
- Adjustable headgear included
- Adjustable PEEP valve designed for Ventumask Basic facilitates some pre-calibrated pressure settings
- Integral oxygen tube



Ventumask Basic for CPAP therapy with Venturi flow driver and adjustable PEEP valve

Requirement		Setting	Flow delivered
O ₂ % approx	PEEP cmH ₂ O	Flow O ₂ L/min	Flow approx L/min
40%	5	12	57
	7.5	12	52
	10	12	47
	12.5	12	42

Performance table subject to changes. Always refer to product labelling.

Code	Description	Size	Box Qty.
CV203S	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Small	5
CV203M/L	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Medium / Large	5
CV203XL	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Extra Large	5



www.intersurgical.com/info/starmed

▶ Video available at www.intersurgical.com

Make an enquiry



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
 T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



Manufacturer:
Intersurgical S.p.A.
Mirandola (MO) Italy

Please think before you print
Save energy and paper. If you must print this information sheet please print it double sided.

IS1.45_INT • Issue 6 06.20